

RAY-BAR X-RAY GLASS POLISH INSTRUCTIONS

X-RAY GLASS "WET" POLISHING INSTRUCTIONS

RAY-BAR X-RAY GLASS POLISH

SPECIALLY FORMULATED FOR GENTLY REDUCING OR ELIMINATING HAZE OR HAIR-LINE SCRATCHES FROM SOFT X-RAY LEAD GLASS SURFACES, OR REFURBISHING OLD EXISTING MEDICAL X-RAY GLASS WINDOWS AND REDUCING SURFACE "GOGGING", DISCOLORATION AND OXIDATION FROM AGE FOR A CLEARER VIEW OF PATIENTS. POLISH AND HAND APPLICATOR ARE INCLUDED ALONG WITH POLISHING INSTRUCTIONS IN THIS KIT. (POLISH WILL NOT REMOVE ETCHES, CHIPS OR DEEP SCRATCHES THAT CAN BE FELT, AND CAN NOT BE UTILIZED ON X-RAY SAFETY GLASS).

DIRECTIONS FOR USE

1. THOROUGHLY CLEAN BOTH SIDES OF GLASS TO BE POLISHED (GLASS MUST BE COMPLETELY CLEANED OF ALL DIRT OR DEBRIS PRIOR TO WET POLISHING)
2. SPRAY GLASS WITH EITHER DISTILLED WATER OR "WINDEX" AND USE WET OR DAMP HAND APPLICATOR COATED WITH POLISH ON A CORNER OR EDGE OF THE GLASS "TEST AREA" FIRST TO EVALUATE RESULTS BY HAND WET-POLISHING METHOD WITH FIRM PRESSURE USING 3" DIAMETER CIRCULAR OVERLAPPING MOTIONS. CONTINUE IF INITIAL RESULTS ARE SATISFACTORY.
3. **ALWAYS KEEP THE POLISH COMPOUND AND APPLICATOR WET AND CLEAN** BY SPRAYING THE GLASS WHILE APPLYING POLISH TO THE GLASS DURING THE MANUAL POLISHING PROCESS.
4. PERIODICALLY CLEAN GLASS DURING POLISHING PROCESS TO CHECK PROGRESS AND RESULT UNTIL SATISFACTORY. FOR BEST RESULTS BE SURE TO POLISH BOTH SIDES OF THE GLASS.
5. CLEAN ALL POLISHING COMPOUND OFF BOTH SIDES OF THE GLASS WITH A VERY SOFT CLEAN COTTON CLOTH OR CLEAN MICRO FIBER CLOTH.
6. THOROUGHLY WASH HANDS AFTER USE.

SPECIAL NOTES

- * NEVER ATTEMPT TO POLISH DEEP SCRATCHES THAT CAN BE "FELT" OR APPEAR CRYSTALLIZED ON THE GLASS SURFACE.
- * ONLY "WET" POLISH GLASS, NEVER RUB DRY POLISH COMPOUND ON GLASS SURFACES.
- * NEVER USE POLISH ON IMPACT RESISTANT X-RAY SAFETY GLASS.
- * NEVER ALLOW MOISTURE OR WATER IN POLISH COMPOUND CONTAINER.
- * RESULTS WILL VARY DUE TO AGE OF GLASS, TYPE OF GLASS COMPOSITION, AND CLEANING COMPOUNDS AND CHEMICALS UTILIZED ON YOUR GLASS BY YOUR MAINTENANCE PERSONS OVER THE YEARS.
- * IF X-RAY GLASS NEEDS REPLACEMENT PLEASE CONTACT RAY-BAR AT **1(800)444-XRAY(9729) OR CONTACT RAY-BAR OF ANY QUESTIONS AT 1(800)444-XRAY(9729) OR VISIT OUR WEBSITE AT : WWW.XRAYGLASS.COM OR WWW.RAYBAR.COM**

Terms of Use: Certain designs and concepts are the exclusive proprietary property and work product of Ray-Bar Engineering provided in good faith as a courtesy for the architectural design of projects specifying, incorporating and utilizing genuine Ray-Bar products and assemblies. Ray-Bar designs are not to be copied, altered, plagiarized or utilized in any competitive or detrimental manner against Ray-Bar as protected by law and/or copyrights. "All Rights Reserved"



JOB NAME:	
LOCATION:	
ROOM NUMBER:	
SHEET ____ OF ____:	

RAY-BAR X-RAY GLASS POLISH

RAY-BAR ENGINEERING CORP.

Toll Free: 1(800) 444-XRAY / 1(800) 444-9729
Telephone: (626) 969-1818
24 Hour Fax: 1(800) 333-XRAY /
1(800) 333-9729
website: www.raybar.com

RAY-BAR
X-RAY
GLASS
POLISH